Gebr. Brasseler GmbH & Co. KG
Trophagener Weg 25
32657 Lemgo - Germany
Telefon +49 (0) 5261 701-400
Telefax +49 (0) 5261 701-580
info@kometmedical.de
www.kometmedical.de

GENIUSadapter
© Gebr. Brasseler GmbH & Co. KG
304384 - Rev 08/2018

DE EN FR ES IT ZH

DE

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EN

Instructions for use and cleaning for GENIUS adapters

1. Indications

The reusable GENIUS rinse adapters are specially developed for the validated cleaning of the GENIUS II Shaver Blades in the cleaning and disinfecting device. Please note that each rinse adapter only fits one type of connection (i.e. Dyonics, Linvatec etc.). The rinse adapters are suitable for Shaver Blades of all diameters.

- To ensure traceability of the MD by means of a UDI, please make sure to keep the packaging of articles without direct marking.

2. Contra indications

Attention: Disposable products cannot be reprocessed. Avoid temperatures of more than 100°C. The rinse adapters are unsuitable for sterilisation.

3. Suitable Shaver Blades

The GENIUS adapter is only suitable for GENIUS II Shaver Blades made by this manufacturer. The validated procedure cannot be transferred to Shaver Blades produced by other manufacturers. Shaver Blades of type GENIUS I cannot be prepared with the GENIUS adapter system.

4. Cleaning and disinfection

- The GENIUS II Shaver Blades are fixed in the rinse adapters according to assembly instructions and connected to the cleaning and disinfecting device.

- The rinse adapter can be attached via ordinary tubes and connections at the mobile unit. Idle connections are to be closed with sealing plugs. For further information please refer to the instruction manual provided with the cleaning and disinfecting device.

- Make sure that the rinse adapters do not come in contact with cleaning and disinfecting agents containing aldehyde and/or enzymes.

- In case of visible contamination, clean the rinse adapters separately in the cleaning and disinfecting device.

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- The rinse adapters can be used for thermal disinfection which has to take place after cleaning, with regard to the A₂ value and in compliance with national provisions.

The reprocessing procedure approved by the manufacturer can be downloaded from our Website www.kometmedical.de.

5. Warnings

- The Shaver Blades must always be securely attached to the rinse adapter. Exchange rinse adapter if this is not possible.

- The rinse adapter always consists of two components (Female Luer-Lock screw + blue plastic body) for each of the tubes of the Shaver Blade.

- The reprocessing according to recommendations does not damage the rinse adapter; wear and tear may, however, be caused by handling.

- Exchange the rinse adapter if the following occurs:

- The trocar and the blade can no longer be fixed in the rinse adapter.

- Material fracture which may jeopardise the success of the preparation.

- The rinse adapter and the Shaver Blades must be protected (by means of a sieve/mesh tray) during the treatment in the cleaning and disinfecting device.

- The Shaver Blades can only be prepared under the condition that all steps of the validated preparation procedures are followed without fail.

6. Safety and liability

- Please observe the instructions in the manufacturer's up-to-date brochure on instrument reprocessing.

- The user is responsible to check the product prior to use whether it is suitable for the intended purpose.

- The user is responsible for the application of the instruments.

- In case of contributory negligence by the user, Gebr. Brasseler partially or totally declines liability for all resulting damages, particularly if these are due to non-observance of our recommendations for use or warnings as well as inadvertent misuse by the user.

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