Gebr. Brasseler GmbH & Co. KG  
Trophagener Weg 25  
32657 Lemgo - Germany  
Telefon +49 (0) 5261 701-400  
Telefax +49 (0) 5261 701-580  
info@kometmedical.de  
www.kometmedical.de

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## Instructions for use and safety recommendations for the application of sterile, disposable corneal rust ring removers for use in ophthalmology

Corneal rust ring removers are instruments for use with battery-powered hand drives. They are made of high-grade stainless steel. Ideally adapted to their indicated use, these instruments are developed, designed and produced with utmost care. They guarantee excellent results provided that they are used in the proper manner. It is therefore essential to observe and comply with the below instructions for use and safety recommendations. Improper use may lead to damage to tissues, premature wear or destruction of the instruments and thus injury to the operator, patient or others.

### 1. Proper use

- All rust ring removers may only be used in appropriate, suitable hand pieces.
- Care should be taken prior to each use that the rust ring removers and their packaging are in immaculate technical condition and perfectly sterile. Do not use the rust ring removers if the instruments or their packaging are in any way damaged or contaminated.
- Insert rust ring removers to their full chucking length into the hand piece/attachment. Insufficiently inserted instruments cause high centrifugal forces and vibrations.
- Before use, check that the instrument is securely fitted in its proper position.
- Do not allow the corneal rust ring removers to come in contact with metal during use. The instruments are not suitable for work on metal.
- Do not use the rust ring remover until the metallic foreign body has been removed from the cornea. Inappropriate use leads to inferior results and increased risk of accidents.
- To ensure traceability of the MD by means of the UDI, please make sure to keep the packaging of articles without direct marking.

### 2. Indication

Corneal rust ring removers are designed to remove the rust ring from the cornea of the eye. A rust ring can develop after a metallic foreign body has penetrated the eye. The foreign body has to be removed before the rust ring remover can be used. If the rust ring is difficult to remove, it is recommended to try again in the next few days because the rust ring tends to "soften".

### 3. Contra indications

In case of the following situations or diseases, the removal of a rust ring with a corneal rust ring remover is only recommended after careful risk assessment and under the condition that facilities for surgery are on standby, should this be necessary.

- Keratoconus and other hereditary or otherwise acquired damage to the corneal tissue.
- Operations of the cornea in the past medical history of the patient (refractive surgery, keratoplasty etc.), especially if these interventions took place less than 12 months ago.
- Other diseases that lead to weakening of the corneal tissue (e.g. caused by laser treatments).

### 4. Contact pressure

Excessive contact pressure is to be avoided because it may lead to further damage to the cornea.

### 5. Discontinue using instruments that show signs of wear during use

Damaged blades, for example by being accidentally dropped, cause vibrations and lead, among other things, to rough surfaces. They have to be discarded immediately, along with any bent or otherwise damaged instruments.

### 6. Reprocessing

The disposable corneal rust ring removers come in sterile packaging. They are not intended for repeated use and therefore do not have to be cleaned/disinfected or sterilized. Prior to dispatch, the corneal rust ring removers were sterilized in their packaging with gamma rays in compliance with the standard ISO 11137 part 1-3. The instruments are designed for single use. During reprocessing, there is a risk of contamination of the patient, the user or third persons.

### 7. Storage

The rust ring removers must be stored at room temperature in a dry place. Avoid exposure to light. Do not use the instruments after expiry of the use-by date.

### 8. Safety and liability

The reuse of these products carries a risk of infection. Due to the wear of the material, a safe, risk-free use can therefore not be guaranteed.

- The user is responsible for checking the products prior to use as to whether they are suited for the intended purpose.
- The operator is responsible for the proper use of the instruments.
- In case of contributory negligence by the user, Gebr. Brasseler partially or totally declines liability for all resulting damages, particularly due to non-observance of our recommendations for use or warnings as well as inadvertent misuse by the user.

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© Gebr. Brasseler GmbH & Co. KG