	<p>Gebr. Brasseler GmbH & Co. KG Trophagener Weg 25 32657 Lemgo - Germany</p> <p>Telefon +49 (0) 5261 701-400 Telefax +49 (0) 5261 701-580 info@kometmedical.de www.kometmedical.de</p>
	

Guide Pins and Screws
© Gebr. Brasseler GmbH & Co. KG
309561 - Rev 2018-05

DE **EN** **FR** **ES** **IT** **ZH**

	<p>Bei beschädigter Verpackung nicht verwenden, Do not use in case of damaged packaging, Ne pas utiliser si l'emballage est endommagé, No utilizar si el embalaje está dañado, Non utilizzare se l'imballo è danneggiato, 如果包装破损请勿使用。</p>
DE	

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EN

Instructions for use and safety recommendations for disposable Guide Pins and Guide Screws in the medical sector

The below instructions and recommendations apply to:
Guide pins for sawing templates
Guide screws for sawing templates

The Guide Pins and Guide Screws are made of high-grade materials. They are developed, constructed and produced with the utmost care. These quality instruments ensure outstanding results and a long service life provided that they are used correctly. It is therefore essential that the below described instructions for use and safety recommendations be observed. Improper use can lead to damage to the tissue, premature wear, destruction of the pins, wires and screws and injury to the operator, the patient or third persons.

Note:
The Guide Pins and Guide Screws are made of implant steel according to the standards DIN 1.4441/316L and DIN 1.4197. The instruments are supplied in sterile or non-sterile packaging, as required. This piece of information is clearly indicated on the label of the instruments.

1. Appropriate use
Guide Pins and Guide Screws may only be used according to the present directions and the instructions for use enclosed with the surgical power systems. Prior to use, make sure that the instruments are absolutely sterile and in perfect technical condition. Do not reuse pins and screws that are in any way damaged. Insert pins and screws into the hand piece until the stop position is reached. Prior to start-up, check that the instrument is securely fitted in its proper position. Guide Pins and Guide Screws may not be used for work on metal or used for leverage. When using saw templates, make sure that the diameter of the Guide Pins and Guide Screws used does not exceed that of the intended holes. Ideally, the diameter of the guide pins and guide screws should be slightly smaller than that of the hole in the template. Improper use leads to unacceptable results and increased risk of injury.
- To ensure traceability of the MD by means of a UDI, please make sure to keep the packaging of articles without direct marking.

2. Indications
Guide Pins and Guide Screws are designed for the fixation of bones at the extremities and in the pelvis area and/ or for example for fixing sawing templates to the bone during endoprosthetic surgery.

3. Contra-indications
All known contraindications for surgery must be generally taken into account. These include, but are not limited to:
a. Diseases that inhibit the healing process
b. Insufficient bone quality
c. Allergies to the metal used. In case of a suspected metal allergy, tests have to be carried out before the start of the treatment.

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Instructions for use
Please read carefully and retain for future reference.

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4. Cleaning, disinfection, sterilization and storage
- For a validated reprocessing method, please visit our website www.kometmedical.de and select “Product Information” on the drop-down menu. Then click on “Reprocessing of Instruments”.
- Instruments supplied in a sterile condition: Do not clean, disinfect or sterilize instruments that come in a sterile condition.
- Instruments supplied in a non-sterile condition have to cleaned, disinfected and sterilized prior to first use. Up to their first use, they should be stored in their original packaging at room temperature, in a dry, dust-free environment.
- The instruments should be stored in hygienically clean blocks, dishes or other suitable containers. The same applies to sterilized instruments and instruments in sterile packages which must be stored away from dust, moisture and any sources of recontamination.
- Pins and screws can be disinfected in the gentle cycle (e.g. Vario TD) of a thermo disinfectior. Only use detergents and disinfectants with proven material compatibility. We recommend deconex 28 AlkaOne (co. Borer Chemie).
- All contact with H₂O₂ (Hydrogen peroxide) as well as detergents and disinfectants containing chlorine or oxalic acid must be avoided, as these may cause pitting and corrosion.
- Use suitable anti-corrosive disinfecting and cleaning agents at all times. Rinse off any disinfecting and cleaning agent immediately after use under running water, as residues of these agents may set off a chemical reaction which could cause corrosion.
- For further information on the use, immersion time and suitability of the disinfecting and cleaning agents please refer to the instructions provided by the manufacturers of these agents.
- During disinfection and sterilisation, make sure that the chosen methods are suitable for the instrument to be treated. Returned instruments are only accepted if they are in clean condition and have been disinfected and sterilised.
- Sterilisation should take place in a steam steriliser (USA 121°C 30 minutes, EU 134°C 5 minutes). Avoid temperatures of more than 180°C.
- Exceeding this temperature would lead to a loss of hardness and therefore a reduced service life.

5. Safety and liability
Do not use the product if the sterile package is damaged. It is the responsibility of the user to check the products prior to use as to whether they are suited for the intended purpose. The user is responsible for the appropriate application of the instruments. In case of contributory negligence by the user, the manufacturer partly or totally declines liability for all resulting damages, particularly due to the non-observance of our recommendations for use or warnings as well as inadvertent misuse by the user. Do not reuse instruments that are intended for single use.

The safe reuse of these products cannot be guaranteed as this poses a risk of infection and/or the products are no longer safe to use.

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