Gebr. Brasseler GmbH & Co. KG
Trophagener Weg 25
32657 Lemgo · Germany

Telefon +49 (0) 5261 701-400
Telefax +49 (0) 5261 701-580
info@kometmedical.de
www.kometmedical.de

Driver
© Gebr. Brasseler GmbH & Co. KG
3 106 12 - Rev 2019-03

DE

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EN

Instructions for use and safety recommendations for pin drivers in the medical sector

The pin drivers are made of high-grade materials. They are developed, constructed and produced with the utmost care. These quality instruments ensure outstanding results and a long service life provided that they are used correctly. It is therefore essential that the below described instructions for use and safety recommendations be observed. Improper use can lead to damage to the tissue, premature wear, destruction of the pin drivers and injury to the operator, the patient or third persons.

Note:

The pin drivers are reusable instruments made of stainless steel. They are supplied non-sterile. The instruments may only be used by adequately trained personnel and according to the intended application. Any other use can lead to the destruction of the instruments and cause harm to the patient.

1. Appropriate use

The pin drivers are used for inserting screws into the bone. The instruments may only be used according to the present directions and the instructions for use enclosed with the surgical power systems. Prior to use, make sure that the instruments are in a perfect technical condition. Do not reuse pin drivers that are in any way damaged (deformed, fractured). The pin drivers may only be used in suitable power systems. Insert pin drivers into the hand piece until the stop position is reached. Prior to start-up, check that the instrument is securely fitted in its proper position in the power system. The pin drivers may not be used for work on metal or used for leverage. Improper use leads to unacceptable results and increased risk of injury. To ensure traceability of the MD by means of the UDI, please make sure to keep the packaging of articles without direct marking.

2. Indications

The pin drivers are designed for the fixation of screws (e.g. GS901.050.032/GS901.055.032) at the extremities of the body.

3. Contra-indications

All known contraindications for surgery must be generally taken into account. Please refer to the instructions for use of the screws used.

4. Cleaning, disinfection, sterilization and storage

- Instruments supplied in a non-sterile condition have to be cleaned, disinfected and sterilized prior to first use. Up to their first use, they should be stored in their original packaging at room temperature, in a dry, dust-free environment.

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Instructions for use

Please read carefully and retain for future reference.

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- For a validated reprocessing method, please visit our website www.kometmedical.de and select "Product Information" on the drop-down menu. Then click on "Reprocessing of Instruments".

- The instruments should be stored in hygienically clean blocks, dishes or other suitable containers. The same applies to sterilized instruments and instruments in sterile packages which must be stored away from dust, moisture and any sources of recontamination.

- Immediately after the operation, remove all instruments without allowing them to dry. Dried residues cause corrosion.

- The pin drivers are suitable for cleaning in an ultrasonic bath. To avoid damage to the instruments, take care that they do not come in contact with one another.

- The pin drivers can be cleaned in the gentle cycle (e.g. Vario TD) of a thermo disinfectant. Only use detergents and disinfectants with proven material compatibility. We recommend Neodisher mediclean forte (co. Dr. Weigert).

- All contact with H₂O₂ (Hydrogen peroxide) as well as detergents and disinfectants containing chlorine or oxalic acid must be avoided, as these may cause pitting and corrosion.

- Use suitable anti-corrosive disinfecting and cleaning agents at all times.

- Rinse off any disinfecting and cleaning agent immediately after use under running water, as residues of these agents may set off a chemical reaction which could cause corrosion.

- For further information on the use, immersion time and suitability of the disinfecting and cleaning agents please refer to the instructions provided by the manufacturers of these agents.

- During disinfection and sterilisation, make sure that the chosen methods are suitable for the instrument to be treated. Returned instruments are only accepted if they are in a clean condition and have been disinfected and sterilised.

- Sterilisation should take place in a steam steriliser (USA 121°C 30 minutes, EU 134°C 5 minutes).

- Avoid temperatures of more than 180°C. Exceeding this temperature would lead to a loss of hardness and therefore a reduced service life.

Make sure that each selected disinfection and sterilization method is suitable for the specific instruments.

The instruments may only be returned in clean, disinfected and sterilized condition.

5. Safety and liability

- It is the responsibility of the user to check the products prior to use as to whether they are suited for the intended purpose.

- The user is responsible for the appropriate application of the instruments.

- In case of contributory negligence by the user, Gebr. Brasseler partly or totally declines liability for all resulting damages, particularly due to the non-observance of our recommendations for use or warnings as well as inadvertent misuse by the user.

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