Komet Medical®  
Gebr. Brasseler GmbH & Co. KG  
Trophagener Weg 25  
32657 Lemgo · Germany  
Telefon +49 (0) 5261 701-400  
Telefax +49 (0) 5261 701-580  
info@kometmedical.de  
www.kometmedical.de

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Instructions for use and safety recommendations for the use of disposable saw blades in bone surgery

Komet Medical® saw blades made of high-grade stainless steel are optimally suited for their specific surgical use. They are designed, constructed and produced with utmost care. These quality instruments assure best working results and long service life provided they are used in the proper manner. Therefore, the following instructions for use and safety recommendations have to be observed. Improper use may damage tissue and may lead to premature wear, destruction of the instruments and injury to the operator, patient or third persons.

### 1. Proper use

- All instruments are to be used in the appropriate handpieces/power systems only.
- Before use, make absolutely sure that the instruments to be used and their packaging are in sterile and in technically sound condition. Discard any damaged instruments and sterile packaging.
- Reciprocating instruments should be inserted as deeply as possible. Instruments which are not inserted to their full chucking length cause high centrifugal forces and vibrations.
- Prior to use make sure that the instrument is safely locked in its proper position.
- The saw blades are not suitable for work on metal (e.g. implants).
- Special notes on the appropriate use of saw blades:
  - The saw blades should be actuated after insertion in the saw guide and before contact is made with the bone.
  - Verify the proper fit and function of the saw blade prior to use in an idle run.
  - When templates are used, only saw blades should be used whose cutting thickness of which does not exceed the thickness of the template. Prior to use on the patient, free mobility of the saw blade has to be assured.
  - Jamming, leverage and bending of the saw blade must be avoided while in the template. Otherwise the saw blade or template will be overheated which leads to jamming of the saw blade or thermal necrosis, or even the risk of fracture.
  - Contact of the saw blade with the template, guiding device or other metallic objects has to be avoided. Otherwise, damage to the saw blade, template and guiding device can occur as well as blunting of the blades in addition to injury to the bone and surrounding tissue.
- Improper use leads to unacceptable results and increased risk of injury.

Do not regrind saw blades. Failure to observe these instructions can lead to the fracture of the instrument and constitutes an increased risk for the patient and the operator. Regrinding of or a mechanical change to the instrument will lead to the loss of the initial cutting properties. The conformity of the instruments with the German Act on Medical Devices is therefore no longer given. Consequently, Gebr. Brasseler/Komet Medical® declines all warranty claims and product liability claims. The user is liable for personal and material damages resulting from reprocessed instruments.

### 2. Indication

Komet Medical® saw blades have been developed for working on bone in the field of orthopaedics and maxillo-facial surgery. The operator is responsible for choosing the appropriate saw blade according to the intended use.

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## Instructions for use

Please read carefully and retain for future reference.

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## 3. Contact pressure

Excessive contact pressure has to be avoided because this might lead to

- damage to the blade surfaces
- chipping of the blades
- reduced service life of the instruments
- excessive heat generation
- thermal necrosis

Excessive contact pressure may lead to overheating which in turn results in thermal necrosis; nicks in the blades may cause undesirable rough surfaces. At the worst, the instrument may break.

## 4. Cooling

Sufficient cooling should be provided when using saw blades. In case of insufficient cooling, the blades will get clogged with bone chips, leading to increased heat generation. At worst, this causes irreversible damage to the bone (thermal necrosis) and reduces service life of the instruments considerably.

## 5. Disposal of blunt and damaged instruments

Check instruments for damages and blunt teeth using magnifying glasses.

Check especially for:

- chipped and blunt (rounded) blades
- damage to the attachment
- bent saw blades

Blunt and damaged instruments must not be reused under any circumstances. Do not regrind saw blades and rasps. Failure to observe these instructions can lead to the fracture of the instrument and constitute an increased risk for the patient and the operator. Regrinding of or a mechanical change to the instruments will lead to the loss of the initial cutting properties. The conformity of the instrument with the German Act on Medical Devices is therefore no longer given. Consequently, Gebr. Brasseler/Komet Medical® declines all warranty claims and product liability claims. The user is liable for personal and material damages resulting from incorrectly reprocessed instruments.

## 5. Safety and liability

- This instrument comes in sterile packaging and does not require processing prior to first use, provided that the sterile packaging is intact. Add patient labels to the patient/hospital record etc. when the sterile packaging is opened for the first time. Komet Medical® regrets being unable to guarantee sterility once the instruments have undergone a reprocessing cycle.
- The instruments may only be returned in clean, disinfected and sterilized condition.

Due to the wear of the material, a safe and risk-free reuse of the product can no longer be guaranteed.

- The user is responsible for checking the product prior to use to ensure that it is suitable for the intended purpose.
- The user is responsible for the use of the instruments.
- In case of contributory negligence by the user, Gebr. Brasseler partially or totally declines liability for all resulting damages, particularly if these are due to non-observance of our recommendations for use or warnings as well as inadvertent misuse by the user.

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© Gebr. Brasseler GmbH & Co. KG